




# HOW TO APPLY FET TAGS

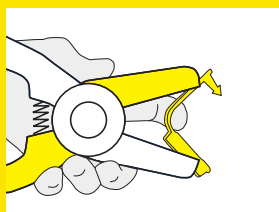


**1.** Place the FET tag into the Z Tags FET Tagger. The male part of the tag should be positioned in the yellow jaw of the tagger, with the female part positioned in the white jaw.

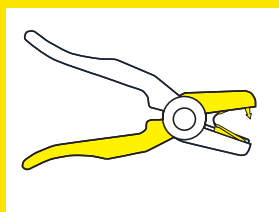

**NOTE:** The image of the tags pin on the yellow jaw is a reminder that the male part of the tag should be positioned there.

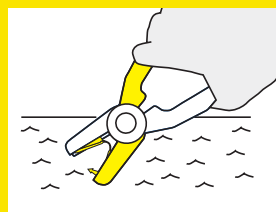

**2.** Dip the tagger holding the ear tag into antiseptic or disinfectant solution.

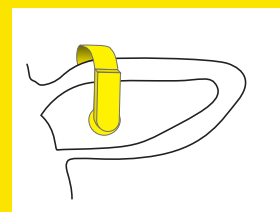

**3.** Apply the FET tag through the ear at one third of the distance from the head avoiding the main blood vessels and the ridges of cartilage. In young animals, remember to allow for ear growth by leaving the FET tag to stick out by one-third of its length from the ear.

Always make sure the male part of the tag is fitted on the back of the ear. After applying the tag, the female part of the tag should be on the inside of the ear.

